


# IMPORTANT: UNPACKING INSTRUCTIONS

## ATOMLAB 960 THYROID UPTAKE SYSTEM

187-600  
187-601

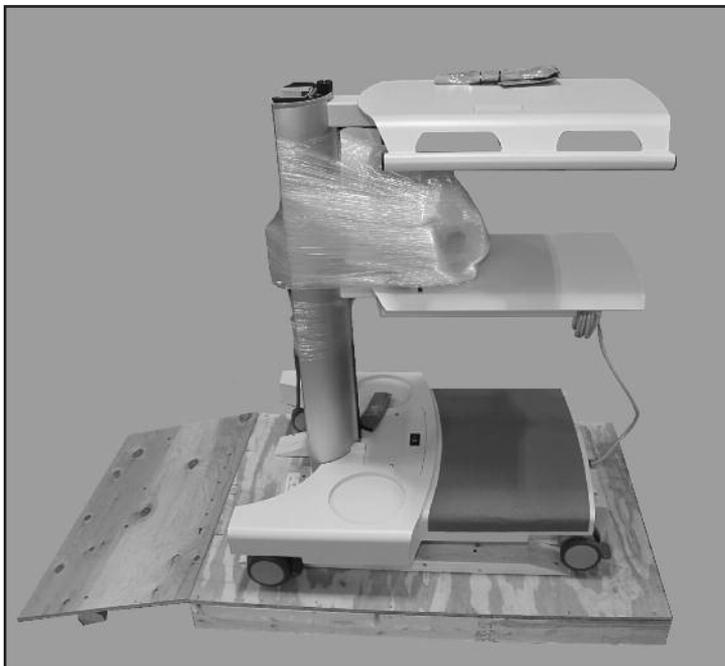

# BIODEX

Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (Int'l 631-924-9000).

Fax: 631-924-9241 Email: [info@biodex.com](mailto:info@biodex.com), [www.biodex.com](http://www.biodex.com)

**This manual covers unpacking instructions for the following products:**

187-600

187-601

**NOTE:** *Two people are required to remove the Thyroid Uptake System from the shipping pallet.*


*Tools Required: Knife, Medium Phillips Head Screwdriver*

**INSTRUCTIONS FOR UNPACKING UPTAKE SYSTEM  
AND RAMP ASSEMBLY**


*(See Figures 1 - 4.)*

The Biodex Thyroid Uptake System you have purchased is shipped in a box on a pallet that contains a ramp as part of the packing material. This ramp can be easily assembled to make it possible to remove the system from the shipping pallet.

1. Cut the straps off the box that secure the Thyroid Uptake System to the pallet.
2. Remove the box from over the contents. If necessary, cut the tall box to remove it.
3. The plywood piece on top of the stand is a ramp to roll the stand out of the box. Slide the 2" x 4" blocks on the end of the ramp under the pallet on the side with the collimator arm.
4. Remove any boxes that are shipped within the uptake stand carton, which include the printer, touch-screen panel and any other parts – put these packages aside for now.
5. Unpack ramp and slide into slots at the front of the pallet..
6. Using a Phillips head screwdriver, remove the block shown in step 2. Lift unit slightly and then remove 2nd block.
7. The stand is shipped with the weight supported by the metal frame on the bottom of the stand (no weight rests on the casters during shipment). Push from the side opposite the collimator arm. The wheels on the base under the arm will connect with the ramp system and slide forward. If there are two people available, each person should grab a handle on the side of the upper shelf. Slide the stand to the edge of the pallet and then carefully roll the stand down the ramp onto the floor.
8. Remove the plastic or protective wrappings and foam from around the arm and stand.
9. Refer to the Atomlab 960 Instruction Manual for Assembly and Use Instructions.

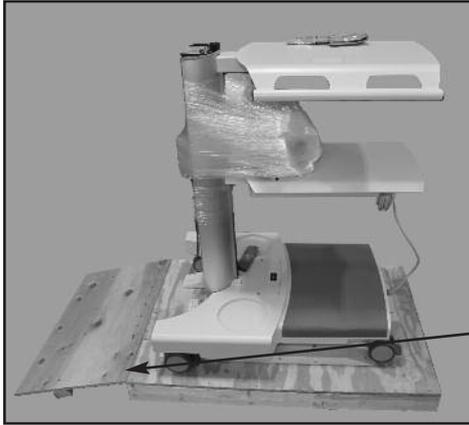


*Figure 1: Unpack ramp, then slide into slots at the front of the pallet.*

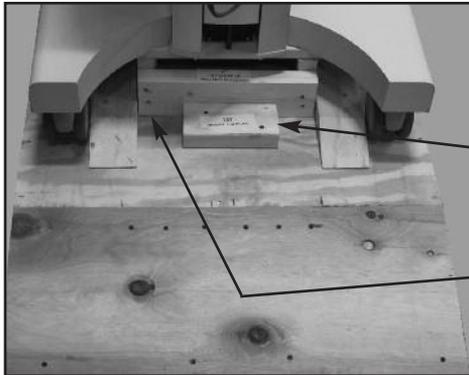


*Figure 2: Using a Phillips Head Screwdriver remove this block first.*

*Lift up slightly, and remove this block.*

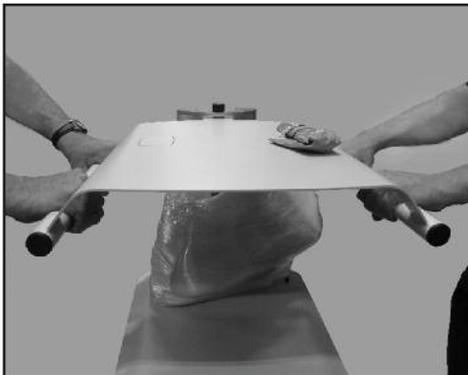

*Figure 3: Using two people, grab the handles on the upper rack to remove unit.*

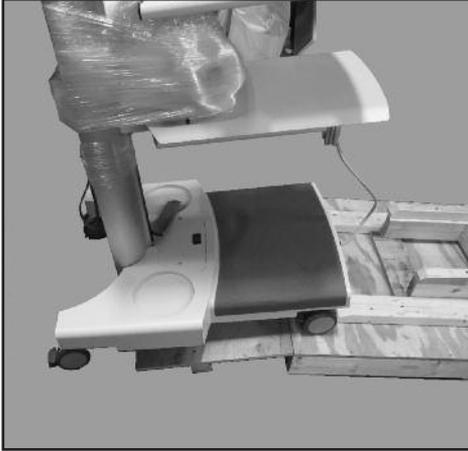

*Figure 4: Using a person on each side, lift the unit slightly while rolling towards and then down the ramp.*

# BIODEX

Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (Int'l 631-924-9000),

Fax: 631-924-9241 Email: [info@biodex.com](mailto:info@biodex.com), [www.biodex.com](http://www.biodex.com)