

# INJECTION STAND

---

OPERATION MANUAL

135-022



**BIODEX**  
Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (Int'l 631-924-9000), Fax: 631-924-9241, Email: info@biodex.com, www.biodex.com

---

# ASSEMBLY AND OPERATION

## Assembly

1. Open the box for the Injection Stand and remove the top foam.
2. Remove the 5 legged-wheeled base and place it on the floor.
3. Remove the straight tube and place the end opposite the knob into the wheeled base.
4. Remove the screw at the top end of the vertical tube near the adjustment knob using a Phillip's Screwdriver. **Save this screw.**
5. Remove the arm rest support section from the box and place it onto the vertical tube. It screws together and is held together with the screw that was removed in Step 4. The vertical tube connects to about the middle of the arm section support. Screw the Phillip's screw in so it is tight.
6. Remove the multi-syringe holder from the packing and slide it into the square stainless panel. Your mobile Injection Stand is now assembled.



*Figure 1. Boxed Injection Stand.*

---

## Instruction for Use

1. To raise or lower the injection stand arm support section, turn the knob on the vertical column counterclockwise to unlock. There is a gas spring inside the tube that will help raise the arm support section up. You may have to pull up on the arm section a little to help raise it. Turn the knob clockwise to lock into position. To lower, turn the knob counterclockwise to loosen and press down on the arm support section until it is in the new position and turn the locking knob clockwise to lock.
2. To adjust the angle of the arm support section, turn the locking knob for the arm support counterclockwise to loosen. Adjust to the angle desired, and turn the knob clockwise to lock.
3. The multi-syringe holder is leaved and you can slide your syringes for the study into this shield until they are ready for use.
4. Standard cleaning solutions can be used to clean the stand and arm support. You can wipe the unit with alcohol.
5. Make sure the wheels are clear of any debris so they roll properly.



**BIODEX**  
Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (Int'l 631-924-9000), Fax: 631-924-9241, Email: [info@biodex.com](mailto:info@biodex.com), [www.biodex.com](http://www.biodex.com)

